



TECHNOLOGY THAT CARES

INTRODUCING

A SAFE WAY TO STORE AND CARRY 'IN-USE' INSULIN

ARKRAY INSULIN COOL POUCH

UNO-Z

*for in-use insulin



NO REFRIGERATION
REQUIRED



ONE YEAR
WARRANTY



ACTIVATED BY
IMMERSING IN WATER



KEEPS INSULIN
COOL AND SAFE



EASY TO CARRY



REUSABLE



www.arkray.co.in

arkray healthcare Pvt. Ltd.

Did you know ?

- Once an Insulin Pen / Vial is opened it should not be kept in fridge. Its stored at Room Temperature (18-30°C).
- If refrigerated / cold insulin is injected, it can be very painful.
- If insulin is exposed to temperature above 30°C, its potency / stability is affected.

| Insulin Brand | Manufacturer | Recommended in-use Insulin storage temperature |
|---------------|----------------|------------------------------------------------|
| Novorapid | Novo Nordisk | Below 30°C |
| Humalog | Eli Lilly | Below 30°C |
| Lantus | Sanofi Aventis | Below 30°C |
| Wosulin | Wockhardt | Below 30°C |
| Lupisulin | Lupin | Below 30°C |
| Levemir | Novo Nordisk | Below 30°C |
| Insupen | Biocon | Below 30°C |

| Estimated days of Insulin stability at different temperatures | |
|---------------------------------------------------------------|------|
| Temperature | Days |
| 25° C | 30 |
| 30° C | 12 |
| 35° C | 4-5 |
| 40° C | 2 |
| 45° C | 1 |
| 50° C | 1/2 |

*Indian J Med Res 130, August 2009, pp166-169

ARKRAY INSULIN COOL POUCH

UNO-Z

*for in-use insulin

KEEPS INSULIN
COOL AND SAFE



Maintains temperature of 18-26°C for upto 45 hours

NO REFRIGERATION
REQUIRED



Hassel-free activation

ACTIVATED BY
IMMERSING IN WATER



Can be activated anytime... anywhere

ONE YEAR
WARRANTY



Assured Quality

REUSABLE



No recurring costs involved

EASY TO CARRY



Freedom to travel with Insulin

EASY 3 STEP PROCESS



Immerse the unactivated inner pouch in cold water for 5-8 mins

Drain the excess water, leave for 20 mins, dab it dry with a towel

Insert the 'In-use' insulin in the inner activated pouch

Insert the inner activated pouch with the 'In-use' insulin in the outer brown pouch and close the zip carefully

For more information contact

Follow us on Social media for interesting info & updates



Diabetes Management Mobile App Scan QR Code to Download (IOS & Android)



ARKRAY



arkray_official



Customer Support (Mon-Fri:9:30 am to 6:30 pm)



WhatsApp: +91 77770 53311 | Email: customersupport@arkray.co.in

ARKRAY Healthcare Pvt. Ltd | 701-702, Opulence, 6th Road, TPS III, Santacruz East, Mumbai 400055, Maharashtra, India